

**ZOLL AED 3™**

# Quick Start Guide



**ZOLL®**

# Welcome

Thank you for selecting the ZOLL AED 3™ as your automated external defibrillator, the most advanced AED in the world today. Follow these simple instructions to set up your AED 3 in just a few minutes. Once setup is complete, note your serial number and log on to [www.zollaed3.com](http://www.zollaed3.com) to register your AED 3 for an extended warranty.

If you have any questions or concerns, please contact ZOLL at +1-978-421-9655 or email us at [techsupport@zoll.com](mailto:techsupport@zoll.com) or contact the authorized distributor you purchased your AED 3 from.

# Contents

## Documentation folder:

- Quick Start Guide
- ZOLL AED 3 Rescue Reference Guide/Poster
- ZOLL AED 3 Operator's Manual

## In the Box:

- ZOLL AED 3
- Documentation folder
- CPR Uni-padz™
- ZOLL AED 3 Battery

# Complete AED Registration


Log on to: [www.zollaed3.com](http://www.zollaed3.com) to:

- Register for your free 2-year extended warranty
- Receive information on setting up your ZOLL Online Account for the upload and management of clinical event data in ZOLL Case Review™


# Setting up your ZOLL AED 3




Remove all contents from the shipping container.

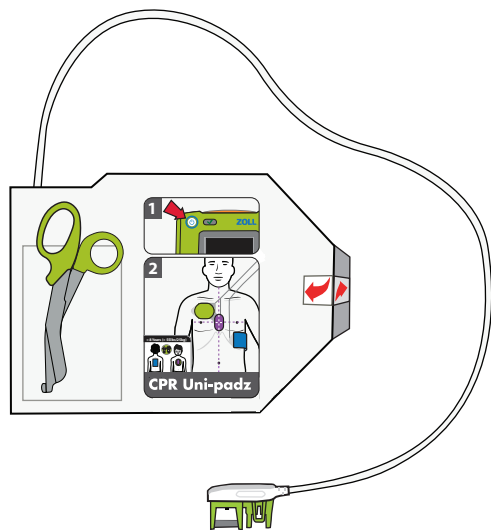


Remove the CPR Uni-padz from their protective box.

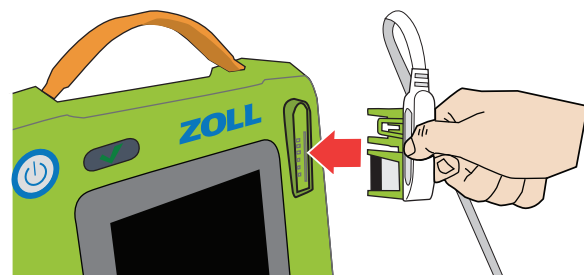


Plug the CPR Uni-padz cable connector into the AED 3 electrode port.

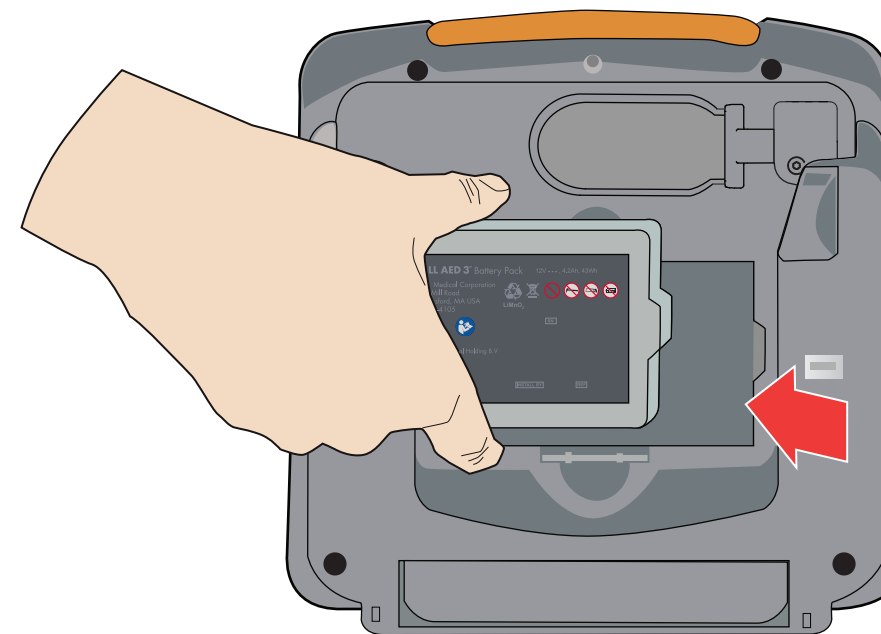

Remove the AED 3 battery from the shipping container. Hold the battery by the side tabs with the label facing out. Insert the battery into the back of the AED until it clicks into place, lining up the notch on the end of the battery to the corresponding shape in the battery well.

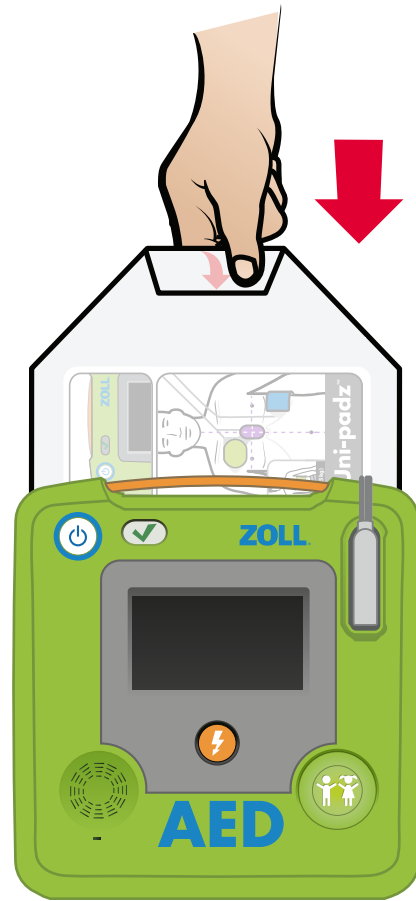

The ZOLL AED 3 will perform an automatic self-test. While the self-test is running, insert the CPR Uni-padz into the back of the AED so that the label on the package faces you.

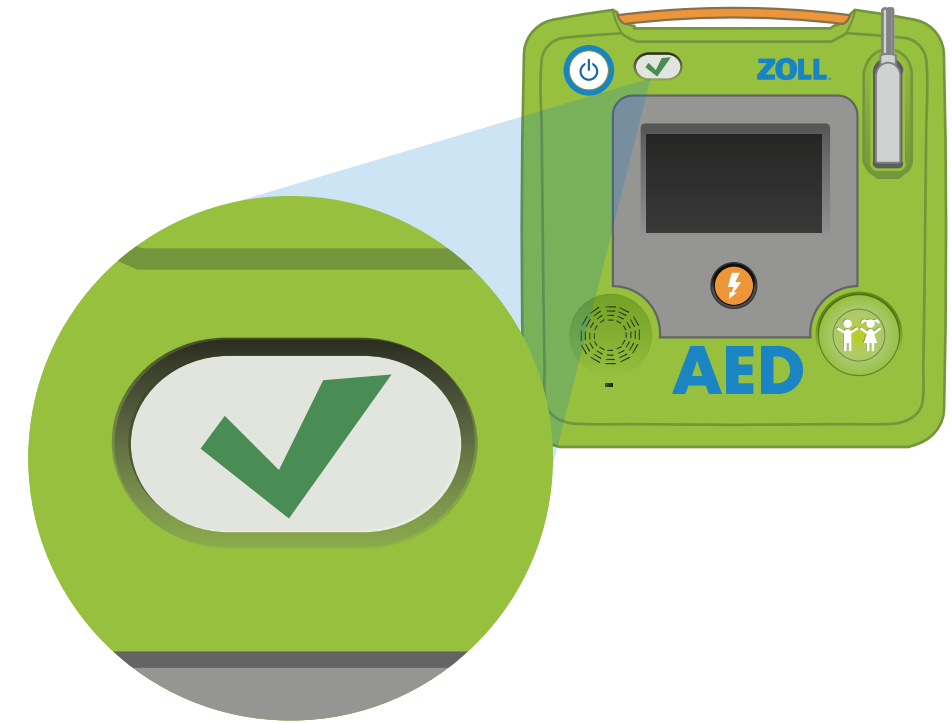

Following the completion of the AED 3 self-test, confirm that it passed by listening for the audible prompt "Unit OK" or looking for the green check symbol in the status indicator window. Your AED 3 is now ready for use. If you have any questions, please refer to the operator's manual included in the documentation folder.

**Questions:**

Contact ZOLL Technical Service or your local authorized ZOLL representative

**ZOLL Technical Service:**

Telephone: +1-978-421-9655

Email: [techsupport@zoll.com](mailto:techsupport@zoll.com)

9650-001753-01 Rev. A

**ZOLL**®